

Drink Detective

Wykrywa w napojach (drinki, piwo, wino, soki itp.) narkotyki powodujące utratę świadomości.

K TEST - KETAMINA

Umieść kroplę napoju na wskazanym polu.

Wynik odczytaj po 10 sekundach.

Białe zabarwienie – wynik pozytywny

Inne zabarwienie – wynik negatywny

B TEST – BENZODIAZEPINY

Umieść 4 krople napoju (w odstępach 5 sekundowych) we wskazanym miejscu na pasku.

Wynik odczytaj po 60 sekundach

Jedna linia – wynik pozytywny

Dwie linie – wynik negatywny



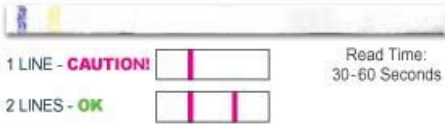
G TEST – GHB

Umieść kroplę napoju na wskazanym polu.

Wynik odczytaj po 10 sekundach.

Niebieskie zabarwienie – wynik pozytywny

Inne zabarwienie – wynik negatywny

HOW TO TEST:	KETAMINE
Put 1 drop of drink on center of test pad.	
 POSITIVE RESULT Red/Orange color change	
May have white center - CAUTION! Any other color - OK	Read Time: 1-10 Seconds
HOW TO TEST:	BENZODIAZEPINES
Put 4 drops of drink on the test pad area below (for best results, leave 5 seconds between drops). Compare to sample results below.	
	TEST HERE with 4 DROPS
1 LINE - CAUTION!	Read Time: 30-60 Seconds
2 LINES - OK	
HOW TO TEST:	GHB
Put 1 drop of drink on center of test pad.	
 POSITIVE RESULT Blue color change	
Blue color change - CAUTION! Any other color - OK	Read Time: 1-10 Seconds